

**URINE CHEMISTRIES**  
**24 Hour Urine Collection and Preservative**  
**(Refrigerate 24 hr urines during collection)**  
 ( X = Preferred Preservative + = Alternate Preservative)

<i>TEST CODE</i>	<i>TEST NAME</i>	<i>None -- Refrigerate During Collection</i>	<i>12M HCL</i>	<i>25 MLS 50% ACETIC ACID</i>	<i>1 GM BORIC ACID</i>	<i>5 GM NA2C03</i>
17KURZ	17-Ketosteroids	X	+			
17CURZ	17-OHCorticosteroids				X	
AMYUR	2 HR Amylase	X				
XYLAZ	5 HR Xylose	X				
5HIURZ	5HIAA	X				
ALDURZ	Aldosterone		+	+	X	
AAQURZ	Amino Acids Quantitative	X				
ARSURZ	Arsenic		X			
ARYAUZ	Arylsulfatase A	X				
B2MURZ	Beta-2 Microglobulin	X				
CAUR	Calcium	X	+	+	+	
CATURZ	Catecholamines	X				
CLUR	Chloride	X				
CROURZ	Chromium		X			
CITURZ	Citric Acid		X		+	
COPURZ	Copper		X			
CORURZ	Cortisol				X	
CREURZ	Creatine	X				
CRCLUR	Creatinine Clearance (also need serum)	X	+	+		
CRUR	Creatinine	X	+	+		
CYSURZ	Cystine	X				
ALAUZ	Delta ALA	X				
GLUR	Glucose	X	+	+	+	
HMTURZ	Heavy Metals		X			
HCYURZ	Homocystine	X				
HVAURZ	Homovanillic Acid	X				
HPRURZ	Hydroxyproline, Total	X				

<i>TEST CODE</i>	<i>TEST NAME</i>	<i>None - - Refrigerate During Collection</i>	<i>12M HCL</i>	<i>25 MLS 50% ACETIC ACID</i>	<i>1 GM BORIC ACID</i>	<i>5 GM NA2C03</i>
IFXUR	Immunofixation	X				
LEDURZ	Lead		X			
LSDUR	LSD, Qualitative	X				
MGUR	Magnesium	X	+	+		
MERURZ	Mercury		X			
MSURZ	Metabolic Screen	Freeze				
METURZ	Metanephrines	X				
MMAURZ	Methylmalonic Acid (MMA)	X				
MHPGUZ	MHPG	X				
MALUR(R)	Microalbumin	X				
MURURZ	Muramidase (Lysozyme)	X				
NIUR	Nitrogen (BUN)	X	+	+	+	
OSTURZ	Osteomark	X				
OXURZ	Oxalate		X			
PEUR	Protein Electrophoresis	X				
PHOSUR	Phosphorus	X	+	+	+	
PBLURZ	Porphobilinogen PROTECT FROM LIGHT!	X				
PORURZ	Porphyrins quantitative PROTECT FROM LIGHT!	X				
KUR	Potassium	X	+		+	
NAUR	Sodium	X	+		+	
TPUR	Total Protein	X			+	
URICUR	Uric Acid	X	+		+	
VMAURZ	Vanillylmandelic Acid (VMA)	X				

